

KN95 FACT SHEET

KN95 MASKS

- Are a type of filtering face mask manufactured and tested overseas that provides a higher level of protection when compared to loose fitting cloth masks, gaiters or nonsurgical facemasks.
- May cause wearers to experience restricted breathing capacity and other side effects including shortness of breath, fatigue, and dizziness. Discontinue use if you experience any of these symptoms.
- Do not require fit testing, training, or medical approval before use. Should not be used in hazardous atmospheres where a fit-tested respirator is required
- Are most effective if there is direct skin to mask contact (No facial hair that interferes with the seal of the mask)



Example of a KN95 mask.



Place your fingertips from both hands at the top of the metal nose clip. Slide fingertips down both sides of the metal strip to mold to the shape of your nose.

How To Wear & Store Your Mask

- Ensure that the mask is secured behind your ears using both ear loops.
- Adjust the metal nose clip using fingers from both hands to mold the clip to the shape of your nose.
- **Re-use of the mask is ok** as long as there is no damage or contamination such as a broken ear loop or a tear/break in the mask material. Discard in regular trash if the mask is damaged, becomes wet or visibly dirty
- Please DO NOT attempt to wash your KN95. With proper care, your KN95 should be reusable for an extended period of time.
- A dry paper bag works well for storing your KN95.